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Instructions for Use

Crystal Temp NE

Non-Eugenol Temporary Cement for Cementing Crowns and Bridges

Crystal Temp NE is an **automix non-eugenol temporary crown and bridge cement** suitable for cementing temporary crowns or bridges as well as for trial cementing of permanent restorations. It shows excellent adhesion to temporary crown and bridge materials, but it can easily removed from prepared tooth stumps. Crystal Temp NE is delivered in MINIMIX syringes so it is very easy and comfortable to use. Due to its non-eugenol formulation it does not influence the setting of resin-based filling and luting materials. The temporary crown and bridge cement is based on ZnO/organic acids. Especially in cases of allergies to eugenol it is strongly recommended to use this eugenol-free type of cement.

Crystal Temp NE meets the requirements of **DIN EN ISO 3107** and the **ADA/ANSI Specification No. 30**.

APPLICATION:

Preparing the MINIMIX-Syringe

Remove the cap of the MINIMIX-syringe and throw it away (**do not use it again!**). It is replaced by a special 1:1 mixing cannula. Turn the cannula 90° until it locks in position. The material is now ready for application.

Note:

Store used syringe with fixed used mixing cannula. Discard the first 2-3 mm of the extruded material. This must be done for each new mix.

Cementing

Crystal Temp NE can be applied directly into the temporary crown or bridge under slight pressure from the mixing cannula.

Use a thin layer to the inside of the temporary crown or bridge and seat not later than **1:30 minutes** (working time) cautiously but firmly over the prepared tooth structures.

Note:

Do not dry stumps, best moisten them with some water. A humid milieu (humid stumps) additionally facilitates to remove the temporary crown or bridge later on.

Crystal Temp NE sets in the mouth within **2-3 minutes**. After Crystal Temp NE has set remove any excess.

Precautions

- Do not use Crystal Temp NE in cases where known allergies to the components exist.
- Do not use Crystal Temp NE if in some cases individual contact allergies may exist.
- In case of contact with skin and eyes rinse immediately with plenty of water and consult a physician if necessary.
- If swallowed seek medical advice.

Storage

Store at 47 - 78 °F (8 - 25 °C). Do not use after expiry date.